

HANDIDAM® INSTRUCTION MANUAL

HandiDam® is manufactured exclusively by Aseptico. It saves time, aids in infection control, and preserves patient goodwill.

Package Contents:

<u>Item No.</u>	<u>Dams Qty.</u>	<u>Sticks Qty.</u>
I-70BM	20 dams	5 sticks
I-70BM-100	100 dams	20 sticks

Instructions:

- 1) Insert one of the sticks into the two holes in the HandiDam® frame as shown in the picture (optional). This serves as a handle when taking radiographs, and comfortably contours the dam to the patient's face.
- 2) Use the dam horizontally as pictured. Punch your hole and use the appropriate clamp, as you would in any standard rubber dam technique.
- 3) When taking radiographs, just fold the dam out of the way to insert film.
- 4) When procedure is complete, discard the HandiDam and frame. Autoclave sticks according to sterilization procedure below and save for re-use.

Sterilization Procedure:

Sterilize the plastic sticks between each patient use. Steam autoclave at 132° C (275° F) for 3 minutes.



LATEX! This product contains latex rubber, which may cause allergic reactions.




The Aseptico HandiDam is a single-use product. Discard after one use.



Protect from light
Store in cool environment
(Room temperature or cooler)

Manufacturer
 
Aseptico Inc
8333 216 St SE
Woodinville
WA 98072
USA


Advena Ltd
Pure Offices
Plato Close
Warwick
CV34 6WE
UK



P.O. Box 1548 • Woodinville, Washington 98072-1548
1-800-426-5913 • 425-487-3157 • FAX: (360) 668-8722

www.aseptico.com • info@aseptico.com

HandiDam®
Pre-framed Rubber Dam



HandiDam®
Pre-framed Rubber Dam
EASY TO USE, CONVENIENT, AND
SAVES TIME



P.O. Box 1548 • Woodinville, Washington 98072-1548
1-800-426-5913 • 425-487-3157 • Fax: 425-487-2608
www.aseptico.com • info@aseptico.com