

HANDIDAM®-LF INSTRUCTION MANUAL

The Aseptico **HandiDam®-LF** dental dam saves time, aids in infection control, and preserves patient goodwill. The latex-free dam is manufactured exclusively with synthetic rubber to avoid patient exposure to latex allergens.

Package Contents:



<u>Item No.</u>	<u>Dams Qty.</u>	<u>Sticks Qty.</u>
I-70LF	20 dams	5 sticks
I-70LF-100	100 dams	20 sticks

Instructions:

- 1) Optional: Insert one of the plastic sticks into the two holes in the HandiDam®-LF frame as shown in the picture. This serves as a handle when taking radiographs, and comfortably contours the dam to the patient's face.
- 2) Use the dam horizontally as pictured. Punch a hole and use the appropriate clamp, as you would in any standard rubber dam technique.
- 3) When taking radiographs, just fold the dam out of the way to insert film.
- 4) When procedure is complete, discard the HandiDam -LF and frame. Autoclave sticks according to sterilization® procedure below and save for re-use.

Sterilization Procedure:

Sterilize the plastic sticks between each patient use. Steam autoclave at 132° C (275° F) for 3 minutes.

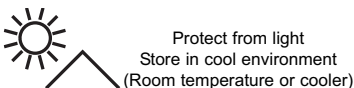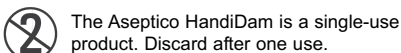

LATEX FREE

HandiDam[®]-LF
Pre-framed
Synthetic Rubber Dam



HandiDam[®]-LF

Pre-framed Synthetic Rubber Dam

**LATEX FREE, EASY TO USE,
CONVENIENT, AND SAVES TIME**

Aseptico[®]

P.O. Box 1548 • Woodinville, Washington 98072-1548
1-800-426-5913 • 425-487-3157 • Fax: 425-487-2608
www.aseptico.com • info@aseptico.com