


SERVICE BULLETIN Handpieces (Latch Type)

Tightening the End Cap

During routine maintenance on handpieces with latch-type heads, it is important to note that the front End Cap is reverse threaded and loosens/ tightens in the opposite direction of standard fasteners. This thread design is preferred because the normal forward clockwise rotation of the handpiece/bur is less likely to loosen the front End Cap.

Always use the supplied wrench to loosen or tighten the front End Cap. Using fingers alone to tighten the End Cap does not provide sufficient torque to ensure proper retention of the endcap.

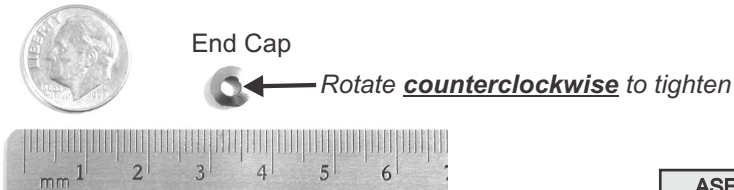

Tighten - counterclockwise
Loosen - clockwise

End Cap wrench,
PN 710129, provided by
Aseptico Inc.
Additional wrenches available
upon request at \$7.00 ea.

**ASEPTICO
HANDPIECES
AFFECTED:**
AHP-60
AHP-61
AHP-62
AHP-62HT
AHP-62M
AHP-63
AHP-66
AHP-88 Series
AHP-85



CAUTION - Always tighten the front End Cap into place with a wrench. Using fingers to tighten the End Cap does not provide enough torque to properly secure it into place.



Aseptico, Inc. • 8333 216th Street SE • Woodinville, WA 98072-1548 • 425-487-3157 • www.aseptico.com


P/N: 420669 • Rev-D • ECO 11984 • 11/2008

SERVICE BULLETIN Handpieces (Latch Type)

Tightening the End Cap

During routine maintenance on handpieces with latch-type heads, it is important to note that the front End Cap is reverse threaded and loosens/ tightens in the opposite direction of standard fasteners. This thread design is preferred because the normal forward clockwise rotation of the handpiece/bur is less likely to loosen the front End Cap.

Always use the supplied wrench to loosen or tighten the front End Cap. Using fingers alone to tighten the End Cap does not provide sufficient torque to ensure proper retention of the endcap.

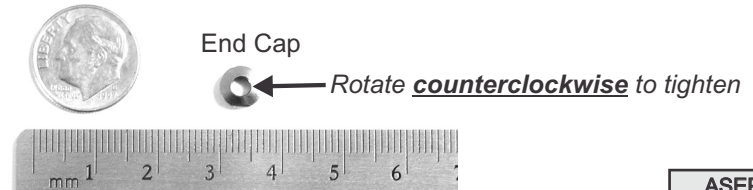

Tighten - counterclockwise
Loosen - clockwise

End Cap wrench,
PN 710129, provided by
Aseptico Inc.
Additional wrenches available
upon request at \$7.00 ea.

**ASEPTICO
HANDPIECES
AFFECTED:**
AHP-60
AHP-61
AHP-62
AHP-62HT
AHP-62M
AHP-63
AHP-66
AHP-88 Series
AHP-85



CAUTION - Always tighten the front End Cap into place with a wrench. Using fingers to tighten the End Cap does not provide enough torque to properly secure it into place.



Aseptico, Inc. • 8333 216th Street SE • Woodinville, WA 98072-1548 • 425-487-3157 • www.aseptico.com

P/N: 420669 • Rev-D • ECO 11984 • 11/2008