

---

---

# ATC-03CF

## Portable Tray Stand

NSN: 6520-01-446-6263

---

---

**COMMAND•AIR**



**OPERATION &  
MAINTENANCE MANUAL**



## **TABLE OF CONTENTS**

<b>Introduction</b> .....	3
Purpose .....	3
Performance Characteristics .....	3
Items Furnished .....	3
<b>Operation Instructions</b> .....	4
Set-Up .....	4
<b>Parts List</b> .....	4
<b>Parts Diagram</b> .....	5

## **TABLE OF ILLUSTRATIONS**

<b>Figure 1 - Parts Diagram</b> .....	6
---------------------------------------	---

## **PURPOSE:**

The Aseptico ATC-03CF Command Air Portable Tray Stand is a lightweight, heavy-duty field system that includes a standard instrument tray with a tripod stand. This tray stand is highly portable and will fit in the mil-spec hard shell case included with the ADC-01CS Command Air Portable Dental Chair for easy deployment.

## **PERFORMANCE CHARACTERISTICS:**

### **Tray and Molded Support:**

Size: 14" x 9.75"

Weight without carrying case: 2.4 lbs

Weight with carrying case: 2.85 lbs

### **Tripod Stand:**

Size (folded): 26" x 4" x 3.5"

Size (open): 25" x 25" x 29" to 57" (Height adjustable)

Weight without carrying case: 5.25 lbs.

Weight with carrying case: 5.65 lbs.

## **ITEMS FURNISHED**


- One (1) AA-50 Tray and Molded Support
- One (1) Tripod stand
- One (1) Soft padded tray carrying case
- One (1) Soft padded tripod stand carrying case
- One (1) Operation and Maintenance Manual

## SET-UP:

1. Remove tray & stand from carrying cases. Unfold tripod legs, slide to the base of the center column, and secure with thumbscrew. Set on floor.
2. Line-up mounting holes on the far end of the molded tray support with the mounting block and thumbscrews on the telescoping center column. Tighten thumb-screws to secure molded tray support to center column.
3. Place stainless steel tray on top of molded tray support. Turn telescoping center column adjustment counterclockwise to loosen, set column to desired height, then turn clockwise to secure.

## PARTS LIST

	Item	Part Number	Quantity
1	Stainless Steel Tray	AA-50	1
2	Molded Tray Support	460104	1
3	Mounting Block	460692-08	1
4	Thumbscrews	510372	2
5	Telescoping Tripod Stand	730252-08	1
6	Center Column Adjustment	-----	1
7	Thumbscrew	-----	1
Not Shown	Storage Pouch, Tray	460796	1
Not Shown	Storage Pouch, Tripod Stand	460797	1

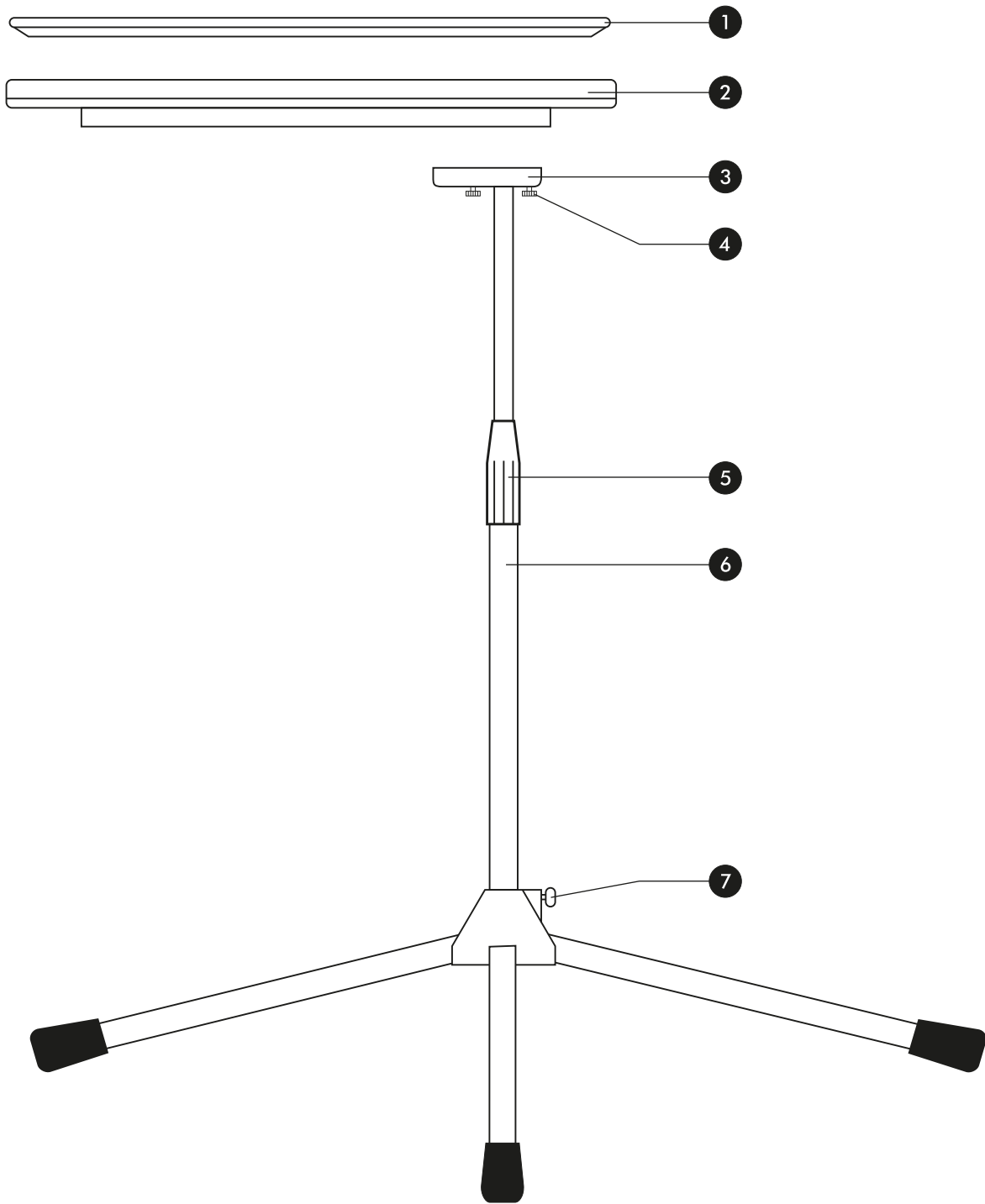

**Aseptico ATC-03CF  
Command Air Portable Tray Stand**

*Figure 1 - Parts Diagram*

P/N 420271  
Rev. B  
ECO 13200  
10/2013



*P.O. Box 1548 • Woodinville, WA 98072-1548  
1-800-426-5913 • 425-487-3157 • Fax: 360-668-8722  
email: [info@aseptico.com](mailto:info@aseptico.com) • Internet: [www.aseptico.com](http://www.aseptico.com)*